

SHADE PEG

Instructions For Use

1. Choose the proper stump shade as prescribed by the dentist, or as dictated by your restoration's needs.
2. Insert the Shade Peg paste into the interior surface of the restoration.
3. Place the restoration on a firing peg.
4. Dry Shade Peg paste under an open oven. Do not close the oven.
5. Allow the restoration to cool.
6. Apply stain and glaze to the restoration as necessary.
7. Fire in a ceramic oven. No vacuum.
8. For Shade Peg Original repeat stain and glaze as necessary.
9. Remove the firing peg
 - a. Shade Peg Original - Lightly sandblast the remainder of Shade Peg from the interior of the crown.
 - b. Shade Peg Release - Steam to remove the remainder from the interior of the crown.

Important Safety Information

- Shade Peg is for laboratory use only. Shade Peg is NOT for intraoral use.
- Wear eye protection when using Shade Peg. Shade Peg contains inorganic refractory materials. If allowed to come into contact with skin or eyes these materials may become an irritant. If allowed to come into contact with skin or eyes, flush with water. If irritation persists, see a doctor.
- Avoid breathing Shade Peg after it has been allowed to dry. Particles in Shade Peg may become airborne after it has dried. It is advised to wear protection when interacting with Shade Peg (e.g. When sandblasting) to protect your respiratory system.