

# Nu:le Coat

Stain & Glaze System



For laboratory & clinical use



Low viscosity for easy application



Transparent glossy coat can be overglazed



Excellent durability



Flexible colour adjustment



Bonding for Peek frame

# Nu:le Coat Stain & Glaze System

For fast, efficient lab or chairside application, a high performance, abrasion resistant Japanese material with a rich colour line up and low viscosity that allows for a thin and uniform application.


- Low viscosity design prevents brush marks.
- With a low curing strain, cracks and lifting are less likely to occur even when the film becomes thicker with repeated coatings.
- Tested up to 50,000 tooth brushing cycles.
- For use with direct and in-direct composite materials.
- Thin hard-coated layers of about 5µm can be created.
- Glaze is ultra transparent and doesn't discolour when light cured.

### Perfect match for:

- Traditional dentures
- Milled dentures, pressed dentures & printed dentures
- Adding glaze after intraoral adjustment
- Composite

### Ultimate Clarity

Nu:le Coat Liquid Clear is a premium quality product which remains 100% transparent when cured and does not distort or tint. This gives complete integrity to the base colours, and the artistic effects beneath the glaze.

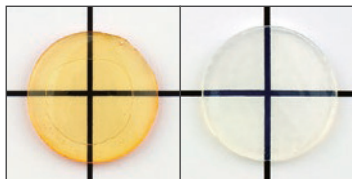

Pellet thickness (immediately after light curing) : 1.0 mm Prototype Clear


### Less Stress

The combination of flexible polyfunctional monomers, and tough cross-linked monomers minimises curing stress. This means the glaze has greater integrity, increased stability and has lower risk of cracking or delamination.



Evaluation of Curing Strain


### Short Working Time

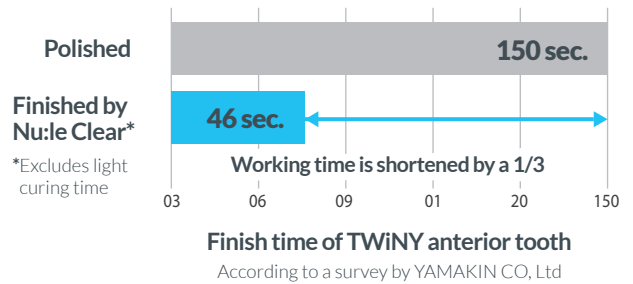

### For Laboratory Use



After shape modification

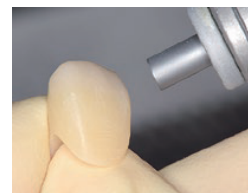

Roughen the surface (Alumina Sandblast etc)



Application of Nu:le Coat (characterising and glazing)



Light Curing (60 sec. by LED CURE Master)



Completion

### For Chairside Use



Filling composite A



After light irradiation, shape modification and roughening of the surface



Application of Nu:le Coat (glazing)



Light irradiation

\* Light irradiators should have an output peak wavelength of 400 ± 20nm with a purple LED




Completion

# Create more aesthetic finishes with Nu:le Coat

## Wide Range of colours for Crown Characterising

White, Grey, Brown, Yellow, Orange and Blue lineups allow for free characterisation of crowns.

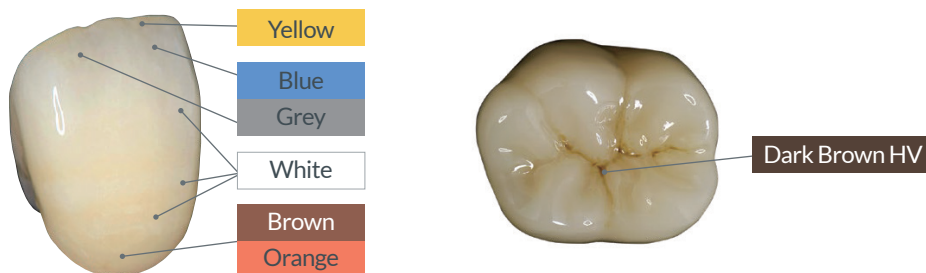


## Shade Plus

There are A plus, B plus, C plus and D plus lineups that allow for fine shade adjustments.



## Gum Plus

There is also a Gum shade line-up (Red, Pink and Black) for denture application or gum expression.

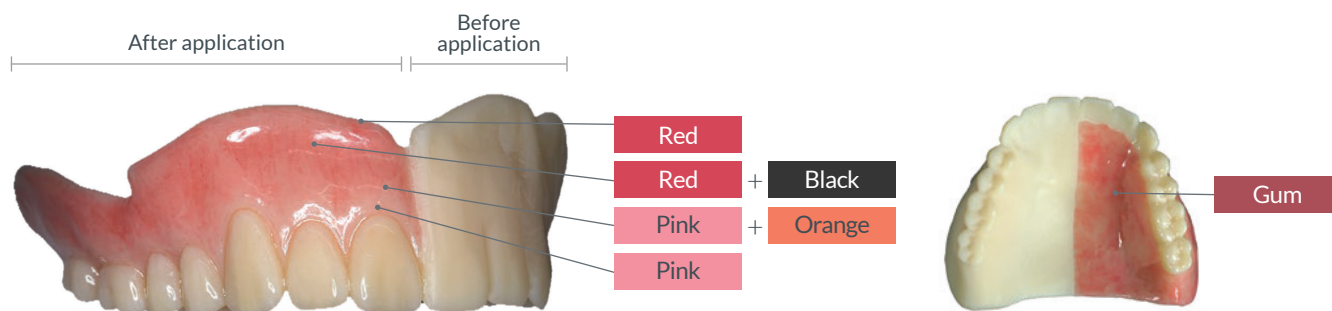

Resin frame characterized with Nu:le Coat Liquid Red, Pink, Black and Orange

# Nu:le Coat



## Nu:le Coat Liquid Single Package

CODE	DESCRIPTION	SIZE	QTY
NULECO18	Nu:le Coat Liquid Clear	6ml	1
NULECO07	White Nu:le Coat Liquid	4ml	1
NULECO08	Grey Nu:le Coat Liquid	4ml	1
NULECO09	Blue Nu:le Coat Liquid	4ml	1
NULECO10	Brown Nu:le Coat Liquid	4ml	1
NULECO11	Yellow Nu:le Coat Liquid	4ml	1
NULECO12	Orange Nu:le Coat Liquid	4ml	1
NULECO13	A plus Nu:le Coat Liquid	4ml	1
NULECO14	B plus Nu:le Coat Liquid	4ml	1
NULECO15	C plus Nu:le Coat Liquid	4ml	1
NULECO16	D plus Nu:le Coat Liquid	4ml	1
NULECO31	Dark Brown HV Nu:le Coat Liquid	4ml	1
NULECO41	Gum Nu:le Coat Liquid	4ml	1
NULECO04	Gum Red Nu:le Coat Liquid	4ml	1
NULECO05	Gum Pink Nu:le Coat Liquid	4ml	1
NULECO06	Gum Black Nu:le Coat Liquid	4ml	1

## Nu:le Coat Gel Single Package

CODE	DESCRIPTION	SIZE	QTY
NULECO17	Nu:le Coat Gel	2ml	1

## Nu:le Coat Full Set

CODE	DESCRIPTION	SIZE	QTY
NULECO03	Nu:le Coat Full Set	-	1
<b>Contents Include:</b>			
	Nu:le Coat Gel	2ml	1
	Nu:le Coat Liquid Clear	6ml	1
	Nu:le Coat Liquid - White, Grey, Blue, Brown, Yellow, Orange, A Plus, B Plus, C Plus, D Plus	4ml	1
	Round Brush	-	1
	Flat Brush	-	1
	Disposal Plate	-	30pk



Scan the QR code to view the Nu:le Coat demonstration videos.

**CE**  
0123

**Controlled Medical Device:** Dental glazing material  
(Colouring material for polymer-based crown & bonding material for dental resin)



**Abbey Dental Supplies**  
4 Imperial Way, Croydon, CR0 4RR  
T: 020 8683 4528  
[www.abbeydentalsupplies.com](http://www.abbeydentalsupplies.com)



**Order Online**

[www.abbeydentalsupplies.com](http://www.abbeydentalsupplies.com)



**Order by phone**

020 8683 4528



**Order by Email**

[sales@abbeydentalsupplies.com](mailto:sales@abbeydentalsupplies.com)



**Order on WhatsApp**

+44 (0) 7586 208031